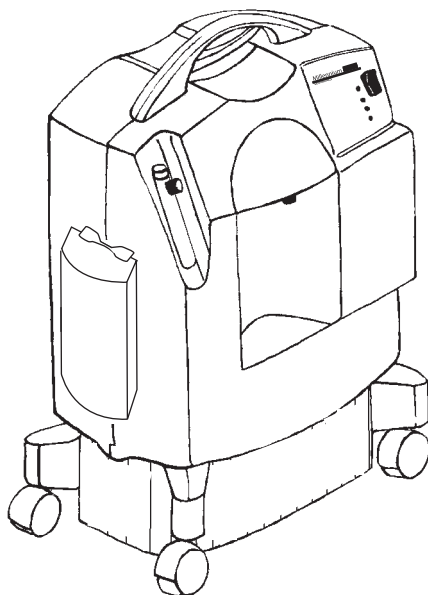
# Millennium M10

---

RESPIRONICS

## *Patient Manual*

*This oxygen concentrator should be used only under the supervision of a licensed physician. Please read and understand this entire manual before using the device.*

# Symbol Glossary

No Smoking

REF

Model number

Attention, consult accompanying documents

SN

Serial Number

Class II equipment

ON (power)

IPX1

Drip proof equipment

OFF (power)

Type BF applied part

## Classification

The Millennium M10 Oxygen Concentrator is classified as:

- IEC Class II Equipment
- Type BF Applied Part
- IPX1 Drip Proof
- Not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- Continuous Operation

## Intended Use

The Millennium M10 Oxygen Concentrator is intended to provide supplemental oxygen to persons requiring oxygen therapy. The device is not intended to be life supporting or life sustaining.

# Specifications

---

Input Voltage:	120 VAC +10%, -10%
Input Frequency:	60 Hz
Average Power Consumption:	600 W
Oxygen Concentration*:	92 ± 4% @ 8-10 LPM 94 ± 2% @ 3-7 LPM 92 ± 4% @ 1-2 LPM
Weight:	24 ± 0.2 kg
Storage/transport temperature:	-34 to 71°C
Operating temperature:	13 to 32°C
Storage/transport humidity:	up to 95%, noncondensing
Operating humidity:	up to 95%, noncondensing
Operating pressure:	69-207 kPa
Altitude:	2438 m

\*Device operation above or outside of the Voltage, LPM, Temperature, Humidity and/or Altitude values specified may decrease oxygen concentration levels.

# Electromagnetic Compatibility

The Millennium M10 is designed to meet the current electromagnetic compatibility requirements. However, if you suspect operation of this device interferes with the normal operation of your TV, radio or other electronic appliances, try:

1. Relocating the appliance or the device until the interference stops or
2. Plugging the device into a different power outlet controlled by a separate circuit breaker or fuse.

# Important Safety Requirements

***Warnings: A warning represents the possibility of harm to the operator or patient***

1. For proper operation, your oxygen concentrator requires unobstructed ventilation. The ventilation ports are located at the rear base of the device and at the side air inlet filter. Always make sure that these areas are not obstructed by items which may impede ventilation (like blankets, foam, curtains, etc.). Do not place the concentrator in a small closed space (such as a closet).
2. Do not remove the covers of the oxygen concentrator. Servicing must be performed by an authorized and trained Philips Respironics home care provider.
3. In the event of an equipment alarm or if you are experiencing any signs of discomfort, consult your home care provider and/or your health care professional immediately.
4. Oxygen generated by this concentrator is supplemental and should not be considered life supporting or life sustaining. In certain circumstances oxygen therapy can be hazardous; any user should seek medical advice prior to using this device.
5. Where the prescribing health care professional has determined that an interruption in the supply of oxygen, for any reason, may have serious consequences to the user, an alternate source of oxygen should be available for immediate use.
6. The use of accessories, transducers, and cables other than those specified by Philips Respironics may result in non-compliance due to increased emissions or decreased immunity of the device. For optimum performance, the oxygen concentrator should only be used with accessories provided by Philips Respironics.
7. Portable and mobile RF communications equipment can affect Medical Electrical Equipment. See the EMC section of this manual for distances to observe between RF Generators and the device to avoid interference.
8. Medical Electrical Equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in this manual.

9. Oxygen vigorously accelerates combustion and should be kept away from heat or open flame. The oxygen concentrator is not suitable for use in the presence of a flammable anesthetic mixture with air, oxygen or nitrous oxide.
10. Do not smoke, allow others to smoke or have open flames near the concentrator when it is in use.
11. Do not use oil or grease on the oxygen concentrator or its components as these substances, when combined with oxygen, can greatly increase the potential for a fire hazard and personal injury.
12. Do not use the oxygen concentrator if either the plug or power cord is damaged. Do not use extension cords or electrical adapters.
13. Do not attempt to clean the oxygen concentrator while it is plugged into an electrical outlet.
14. Operation of the oxygen concentrator above or outside of the Voltage, LPM, Temperature, Humidity and/or Altitude values specified may decrease oxygen concentration levels.
15. Your home care provider is responsible for performing appropriate preventive maintenance at the intervals recommended by the device manufacturer.

***Cautions: A caution represents the possibility of damage to the equipment.***

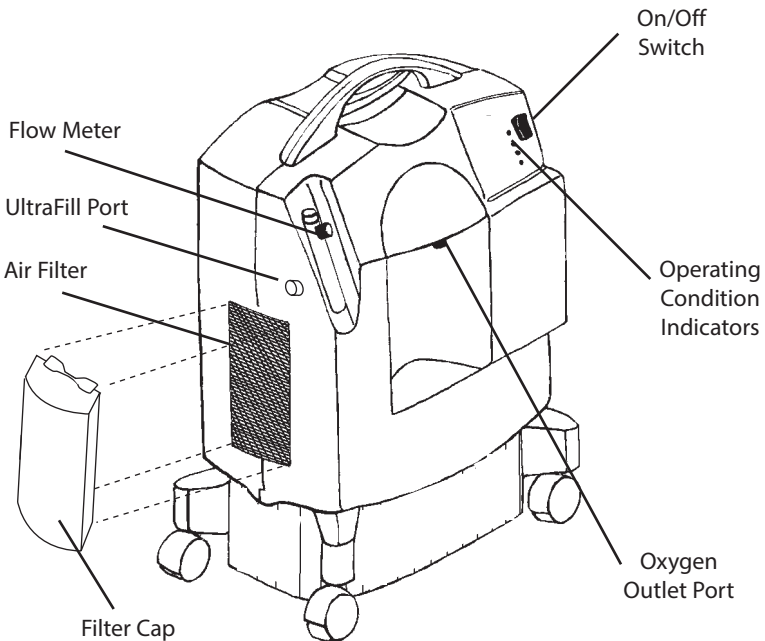
1. Do not operate the oxygen concentrator without all the filters in place. The filters must be totally dry before operating the device.
2. Do not place liquids on or near the oxygen concentrator.
3. If liquid is spilled on the oxygen concentrator, turn the power off and unplug from the electrical outlet before attempting to clean up the spill. Call your home care provider if the device does not continue to work properly.
4. The device should not be used adjacent to or stacked with other equipment. For more information, contact your home care provider.

# Introduction

Your health care professional has determined that supplemental oxygen is of benefit to you and has prescribed an oxygen concentrator set at a specific flow setting to meet your needs. DO NOT change the flow settings unless your health care professional tells you to do so. The Millennium M10 oxygen concentrator is intended to provide supplemental oxygen to persons requiring oxygen therapy. This device is not intended to be life supporting or life sustaining. The Millennium produces concentrated oxygen from room air for delivery to a patient requiring low flow oxygen therapy. The oxygen from the air is concentrated using a molecular sieve and a pressure swing absorption process. Your home care provider will show you how to operate the concentrator and will be available to answer any questions. You should contact him or her in the future if you have any questions or problems.


Millennium M10 models that are compatible with the UltraFill Oxygen Filling Station will include an additional connection port for connection to the UltraFill Oxygen Filling Station. The performance of the Millennium M10 is not affected when the port is connected to the UltraFill Oxygen Filling Station. The port is for connection to the UltraFill Oxygen Filling Station **only** and not for connecting the patient oxygen cannula. The patient oxygen cannula is connected to the Millennium M10 as described in the **Operating Instructions** section in this manual. Please see the UltraFill manual for connection and operating instructions regarding the UltraFill Oxygen Filling Station.

# Parts of Your Concentrator


# Operating Instructions

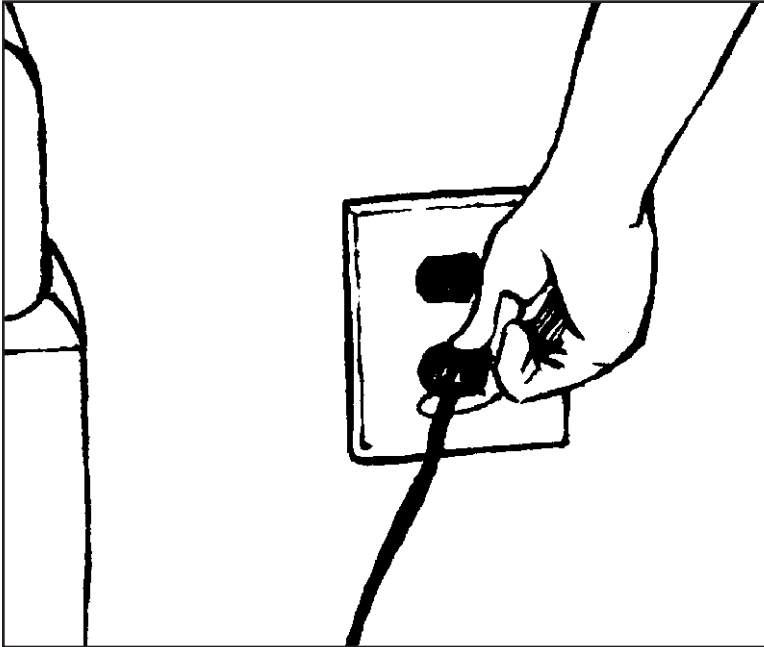1. Select a location that allows the oxygen concentrator to draw in room air without being restricted. Make sure that the oxygen concentrator is at least 8 cm away from walls, furniture, and especially curtains that could impede adequate airflow to the device. Do not place the oxygen concentrator near any heat source.
2. After reading this entire manual, plug the power cord into an electrical outlet.

**WARNING:**  
DO NOT USE EXTENSION  
CORDS OR ELECTRICAL  
ADAPTERS.

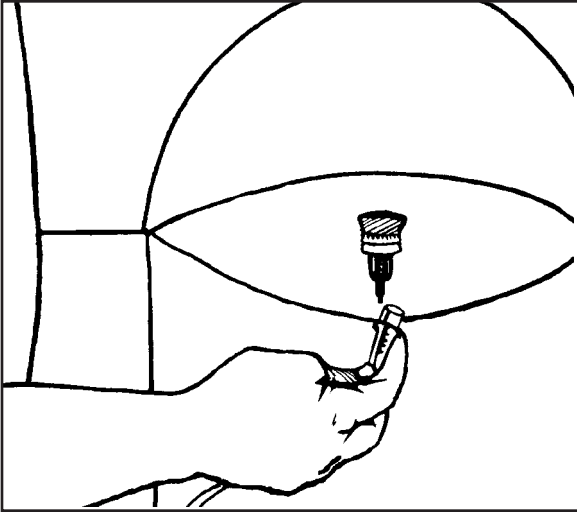3. Connect the oxygen tubing (cannula) to the oxygen outlet port.

4. Press the power switch to the On [ I ] position. Initially, all the indicator lights will turn on and the audible alarm will sound for a few seconds. After that time, only the green indicator light should remain lit.

5. After turning on the oxygen concentrator, allow at least 10 minutes for it to warm up.
6. Adjust the flow to the prescribed setting by turning the knob on the top of the flow meter until the ball is centered on the line marking the specific flow rate.
7. Be sure oxygen is flowing through the cannula. If it is not, refer to the Troubleshooting section.
8. Put the cannula on as directed by your home care provider.
9. When not using the device, press the power switch to the Off [ O ] position.

## Accessory Equipment

Use only the following Philips Respironics accessories with the Millennium M10:

- Part Number 1014653      Cannula and Tubing

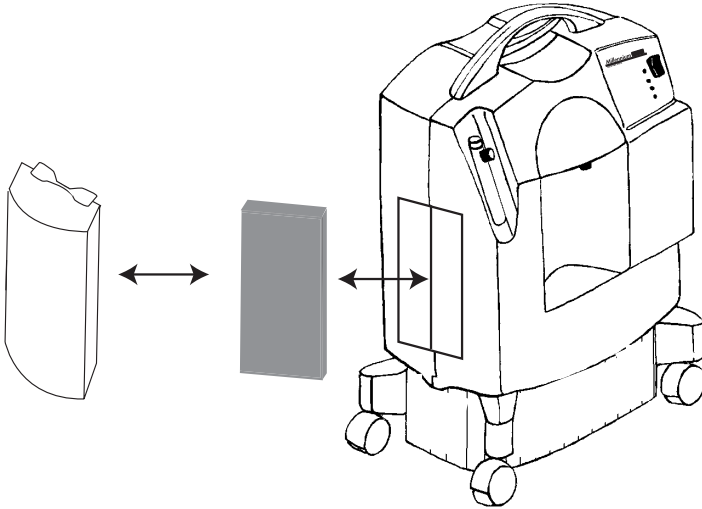
Contact your home care provider if you have questions concerning the use of these accessories.

# Maintenance

**Warning: It is important to unplug the device before any maintenance or cleaning is performed.**

## Cleaning and Changing the Air Inlet Filter

Cleaning the air inlet filter is the most important maintenance activity that you will perform and should be done at least once a week.

1. Remove the filter cap.
2. Remove the filter from the cabinet.
3. Visually inspect the filter for damage, such as holes or tears.
4. If damaged, replace with a new filter. Contact your health care provider for a new filter if necessary.
5. Rinse and wash the filter in warm water. A mild detergent may be used if rinsed thoroughly.
6. Squeeze out the excess water and allow the filter to air dry. The filter should be completely dry before using again. Excess moisture may impair the proper operation of the device.
7. Visually inspect the filter after cleaning. Make sure it is not damaged or clogged.
8. Reinsert the filter on the cabinet.
9. Reinstall the filter cap.


## Cleaning the Device

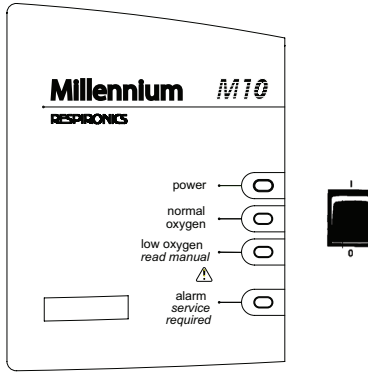You should also clean the exterior case of the oxygen concentrator. You may use a damp cloth to wipe down the exterior case. If medical disinfectants are used, be sure to follow the manufacturer's instructions.

# Power Interrupt Alarm

If the device is turned on but not operating, the alarm condition will be activated. This is often the result of either the oxygen concentrator not being plugged in or a power failure. This feature is powered by an internal 9V battery.

To test the condition of the battery you should test this alarm state often by unplugging the device and turn the power switch on. If the alarm condition is not activated, you should contact your home care provider to replace the battery. If the device shuts down because of a power failure, turn the power switch off for two minutes before restarting to allow the system to depressurize itself.

## Operating Condition Indicators and Alarms

- **Power:** The GREEN indicator illuminates when the device is turned on.
- **Normal Oxygen** (Model M10605 only): The extra GREEN indicator illuminates when the device is delivering a normally expected percentage of oxygen. The illumination of this indicator is delayed for several minutes after start up to allow the system to stabilize.
- **Low Oxygen:** The YELLOW indicator illuminates continuously\* to alert the user when some corrective action is required. Following the troubleshooting guide in this manual, this action can normally be performed by the user. Call your home care provider if the troubleshooting actions fail to end this alert condition.
- **Alarm:** The RED indicator illuminates continuously and the audible alarm sounds continuously if the device detects a system malfunction. If this happens when the device is plugged in, you should turn off the device and call your home care professional immediately.

\* Units may have intermittent RED indicator and sound an audible alarm accompanying the YELLOW indicator under a Low Oxygen condition.

# Troubleshooting Guide

<b>Problem</b>	<b>Probable Cause</b>	<b>Solution</b>
The device is not working when it is turned on; the alarm condition (red light) is activated and the power indicator (green light) is not illuminated.	The power cord plug is not properly inserted into the electrical outlet. No power from the electrical outlet. Internal part failure.	Make sure the device is properly plugged in to the electrical outlet. Check your household outlet fuse or circuit breaker. Contact your home care provider.
Low oxygen (yellow light) activated.*	The airflow to the device is impeded or blocked. The flowmeter knob is completely closed. The oxygen tubing is kinked and blocking the delivery of oxygen.	Check the condition of the air inlet filter and clean or replace as necessary. Remove any items that appear to be blocking the airflow into the device. Turn flowmeter knob counterclockwise to center ball on prescribed LPM Flow. Check to see that the tubing is not kinked or blocked. Replace if necessary.
Alarm condition (red light) activated, when the device is on with the power indicator (green light) on.	Internal system failure.	Contact your home care provider immediately.
Limited oxygen flow to the user.	Faulty oxygen tubing, cannula. Poor connection of device accessory.	Inspect and replace the items if necessary. Ensure that all connections are free from leaks.
Alarm is weak or not sounding when the power cord is un-plugged and power switch is on.	The 9V battery is weak.	Contact your home care provider.

\* Units may have intermittent RED indicator and sound an audible alarm accompanying the YELLOW indicator under a Low Oxygen condition.

# EMC Information

Guidance and Manufacturer's Declaration – Electromagnetic Emissions		
This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure that it is used in such an environment.		
Emissions Test	Compliance	Electromagnetic Environment - Guidance
RF Emissions CISPR 11	Group 1	The Device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class B	The Device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/Flicker Emissions IEC 61000-3-3	Complies	

Guidance and Manufacturer's Declaration – Electromagnetic Immunity			
This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	±6 kV Contact ±8 kV Air	±6 kV Contact ±8 kV Air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical Fast Transient/Burst IEC 61000-4-4	±2 kV for Power Supply Lines ±1 kV for Input/Output Lines	±2 kV for Power Supply Lines Not Applicable	Mains power quality should be that of a typical home or hospital environment.
Surge IEC 61000-4-5	±1 kV Line to Line ±2 kV Line to Ground	±1 kV Line to Line ±2 kV Line to Ground	Mains power quality should be that of a typical home or hospital environment.
Voltage Dips, Short Interruptions and Voltage Variations on Power Supply Input Lines IEC 61000-4-11	<5% $U_T$ (>95% Dip in $U_T$ ) for 0.5 Cycle  40% $U_T$ (60% Dip in $U_T$ ) for 5 Cycles  70% $U_T$ (30% Dip in $U_T$ ) for 25 Cycles  <5% $U_T$ (>95% Dip in $U_T$ ) for 5 Seconds	<5% $U_T$ (>95% Dip in $U_T$ ) for 0.5 Cycle  40% $U_T$ (60% Dip in $U_T$ ) for 5 Cycles  70% $U_T$ (30% Dip in $U_T$ ) for 25 Cycles  <5% $U_T$ (>95% Dip in $U_T$ ) for 5 Seconds	Mains power quality should be that of a typical home or hospital environment. If the user of the Device requires continued operation during power mains interruptions, it is recommended that the Device be powered from an uninterruptible power supply or battery.
Power Frequency (50/60 Hz) Magnetic Field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical home or hospital environment.
Note: $U_T$ is the A.C. mains voltage prior to application of the test level.			

### Guidance and Manufacturer's Declaration – Electromagnetic Immunity

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the Device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  <b>Recommended separation distance</b>  $d = 1.2\sqrt{P}$  $d = 1.2\sqrt{P}$ 80 MHz to 800 MHz $d = 2.3\sqrt{P}$ 800 MHz to 2.5 GHz  where $P$ is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and $d$ is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup> Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m 80 MHz to 2.5 GHz	
Note 1:	At 80 MHz and 800 MHz, the higher frequency range applies.		
Note 2:	These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.		
a	Field strength from fixed transmitters such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Device is used exceeds the applicable RF compliance level above, the Device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Device.		
b	Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.		

### Recommended Separation Distance Between Portable and Mobile RF Communication Equipment and this Device

The device is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of this device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communication equipment (transmitters) and this device as recommended below, according to the maximum output power of the communications equipment.

Rated Maximum Output Power of Transmitter (Watts)	Separation Distance According to Frequency of Transmitter (meters)		
	150 kHz to 80 MHz outside ISM Bands $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

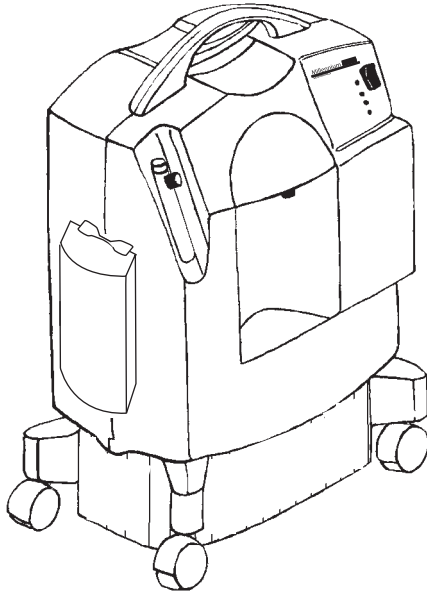
Note 1:	At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
Note 2:	These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

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Respironics Inc.